




# Set up in three easy steps

## Mostcare Up


Step one: Connect Mostcare Up

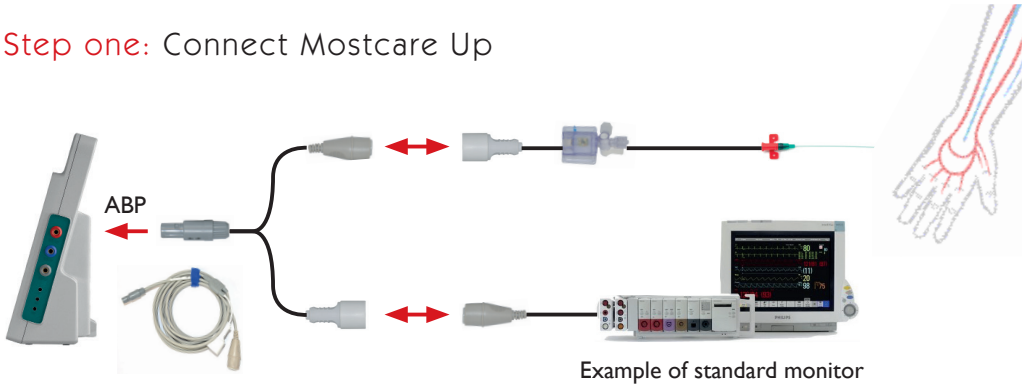

Connect the Y cable between the bedside monitor and the ABP transducer

or

Connect the analogue cable between Mostcare Up and the bedside monitor

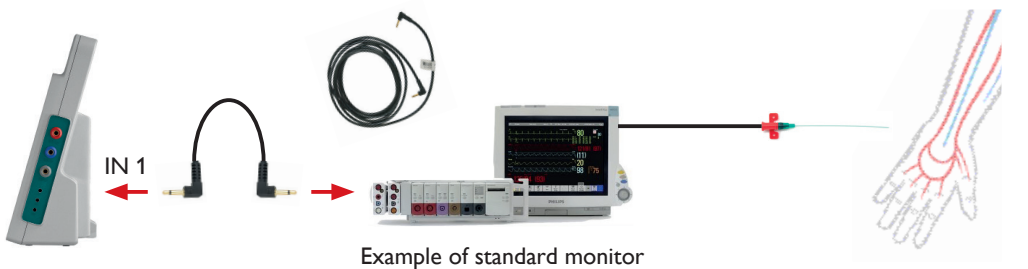

## Scan with your smartphone

to view online or visit [vygon.co.uk/mostcareupevaluation/](https://www.vygon.co.uk/mostcareupevaluation/)

Tel: 01793 748800 Copyright Vygon (UK) Ltd 2022 Content correct as of: 01/2022 Code: V00612 v4




## Step two: Enter patient data

1. Press New monitoring
2. Enter patient weight and height
3. Select ABP transducer and tick the Y cable box if applicable
4. Check other settings and press Begin monitoring



## Step three: Zero the transducer

- Place the transducer at the level of the phlebostatic axis
- Open the transducer stopcock to measure the atmospheric pressure
- Touch the curve on the screen and press **Zero ABP**
- Zero the primary monitor and re-establish trace

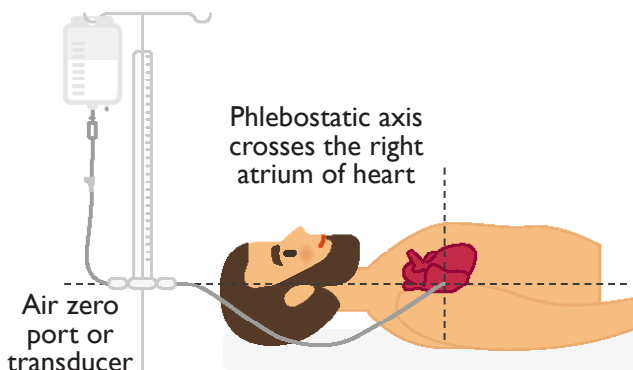

Scan with your smartphone

to view online or visit [vygon.co.uk/mostcareevaluation/](https://www.vygon.co.uk/mostcareevaluation/)

Tel: 01793 748800 Copyright Vygon (UK) Ltd 2022 Content correct as of: 01/2022 Code: V00612 v4

